WSAUD A/S
Nymoellevej 6, DK-3540 Lyngby, Denmark
<https://global.widex.com>

Manual no.:
9 514 0512 080 03
Issue:
2020-11





WIDEX **REMOTE LINK**[™]

USER INSTRUCTIONS
REMOTE LINK

GB
SC
DE
DK
ES
FR
IT
JP
PT
US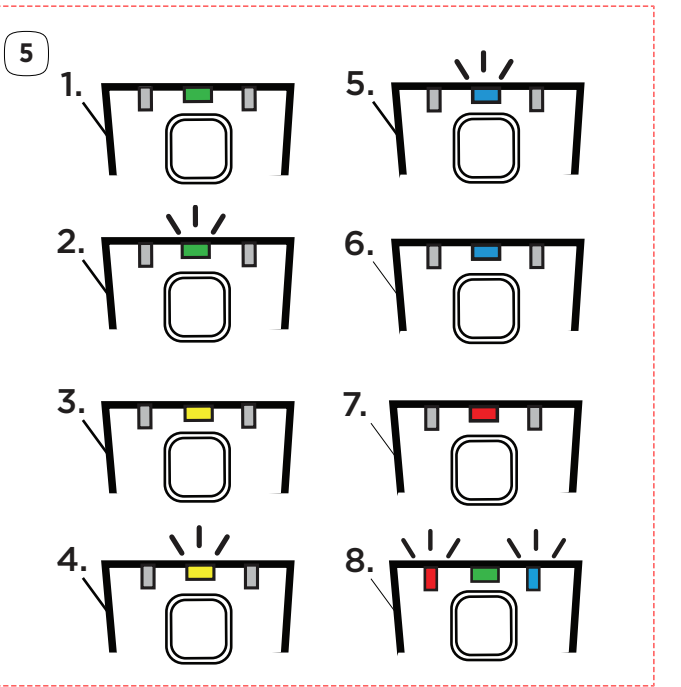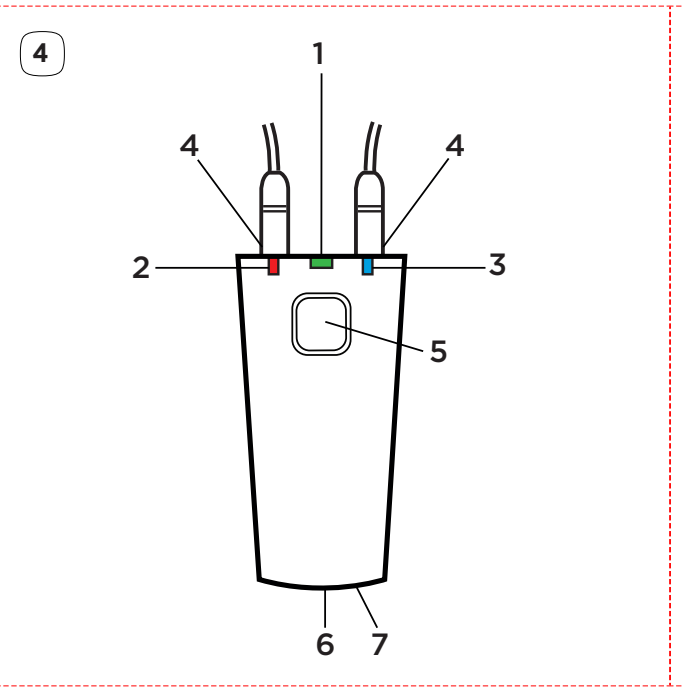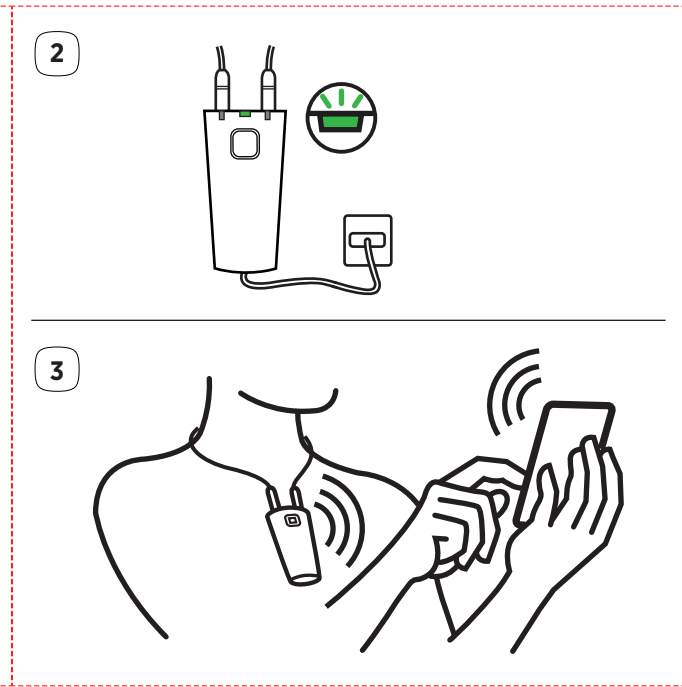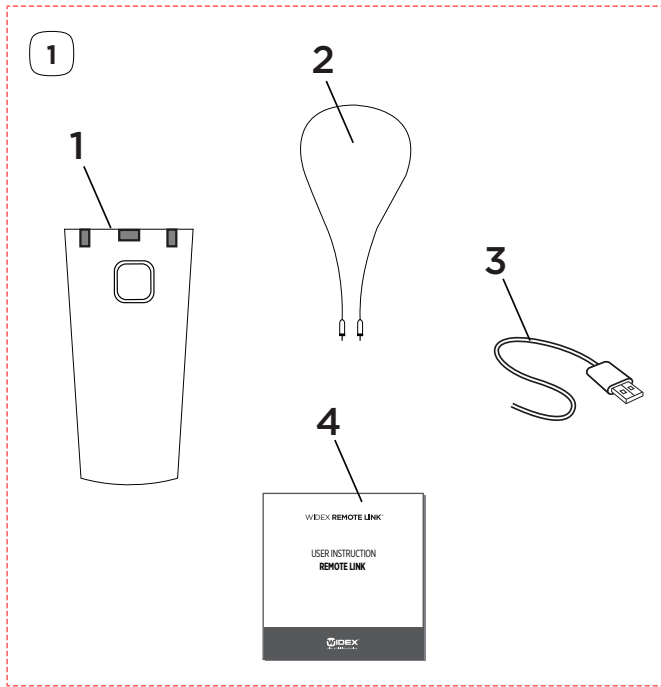

GB	USER INSTRUCTIONS	4
SC	用户使用手册	14
DE	BEDIENUNGSANLEITUNG	22
DK	BRUGSANVISNING	32
ES	INSTRUCCIONES DE USO	41
FR	MODE D'EMPLOI	51
IT	LIBRETTO D'ISTRUZIONI	61
JP	取扱説明書	71
PT	INSTRUÇÕES DE UTILIZAÇÃO	82
US	USER INSTRUCTIONS	92
	FCC AND ISCED STATEMENTS	102
	LICENSE AGREEMENT	105

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The device — see illustration 4:

- 4.1 Centre LED
- 4.2 LED for the right hearing aid
- 4.3 LED for the left hearing aid
- 4.4 Neckloop connectors
- 4.5 Push button (on/off switch and button for pairing)
- 4.6 Micro USB port for charging
- 4.7 Reset pinhole button

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NOTE

Please charge the battery fully before using it the first time. While charging, the centre LED on the device will be flashing. When the device has been fully charged, the centre LED will stop flashing.

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PACKAGE CONTENTS AND DESCRIPTION OF THE DEVICE

Package contents — see illustration 1:

- 1.1 Device
- 1.2 Neckloop
- 1.3 Micro USB cable
- 1.4 User instructions

The device — see illustration 4:

- 4.1 Center LED
- 4.2 LED for the right hearing aid
- 4.3 LED for the left hearing aid
- 4.4 Neckloop connectors
- 4.5 Push button (on/off switch and button for pairing)
- 4.6 Micro USB port for charging
- 4.7 Reset pinhole button

IMPORTANT SAFETY INFORMATION

Warnings

Read this page carefully before you start using this device.

Do not use the device in mines or other areas with explosive gases.

Do not expose to extreme temperatures or high humidity.

When the device is connected via micro USB cable to external mains-operated equipment like a charger, PC, audio source or similar, this equipment must comply with IEC 62368-1, IEC 60601-1 or equivalent safety standards.

Any change or modification to the device that has not been expressly approved by the manufacturer could void the user's authority to operate the device.

Do not use the product at altitudes above 2,000 metres, as extremely low air pressure may result in an explosion or the leakage of flammable liquid or gas.

Do not dispose of the product with its battery into fire or a hot oven.

The device should be stored and transported within the temperature and humidity ranges of -10°C to +50°C (14°F to 122°F) and < 90% rH.

The device is designed to operate from 5°C (41°F) to 40°C (104°F).

For more information about the device, visit: www.widex.com.

INTENDED USE

The device is intended to connect Widex fitting software to your W-Link hearing aid* for fitting purposes.

*hearing aid with wireless connectivity

SETTING UP AND USING THE DEVICE

Charging the device — see illustration 2

Using the micro USB cable, connect the device to your computer or to a USB adaptor connected to a power outlet.

We recommend using a 5V 500mA USB adaptor.

NOTE

Please charge the battery fully before using it the first time. While charging, the center LED on the device will be flashing. When the device has been fully charged, the center LED will stop flashing.

Turning the device on and off

To turn on the device, press and hold the push button for a few seconds until the LEDs light up.

To turn off the device, press and hold the push button for a few seconds until all the LEDs are off.

Connecting the device to the WIDEX REMOTE CARE app

When you receive the device, your hearing care professional will have prepared it for you. After you receive the device, it has to be connected to the WIDEX REMOTE CARE app before you use it. If the device has already been connected to the app, you can skip this section.

To connect the device to the app, do as follows:

1. Download the WIDEX REMOTE CARE app
2. Open the app
3. Follow the instructions in the app

NOTE

When connecting the device to your app, please make sure that your smartphone is not connected to any other devices via Bluetooth.

If the device does not turn on, please recharge it before trying again.

Using the device — see illustration 3

Turn on the device and place it around your neck.

Make sure that your smartphone is connected to the internet.

Go to the WIDEX REMOTE CARE app on your smartphone.

Your hearing care professional will connect to your hearing aids from the clinic via the fitting software and the fitting device. Your hearing care professional is now in control of your hearing aids.

Follow the instructions from your hearing care professional through the app.

When the remote fit has been completed, you can disconnect and turn off the device.

Visual indicators — see illustration 5

The following table shows the various LEDs and what they indicate:

LED	Indication
5.1	The device is on and the battery charged
5.2	The device is charging
5.3	The battery is running low
5.4	The battery is very low
5.5	The device is in pairing mode
5.6	The device is connected via Bluetooth (only shown shortly after the device has been connected)
5.7	The device is malfunctioning. Please return the device to your hearing care professional
5.8	The device is communicating with your hearing aids

MAINTENANCE

It is important that you keep the device clean and disinfected before use.

- Wipe the device and neckloop with a damp cloth and clean the accessories with a dry cloth. Do not use any chemicals.
- Do not immerse the device or neckloop in water or any other liquid.

- While the device is not in use, keep it in a dry location out of reach of children and pets.
- Never try to open or repair the device yourself. In case of malfunctions, please contact your hearing care professional.

TROUBLESHOOTING

Problem	Possible cause	Solution
The device does not work	a: The battery is low b: The device has not been paired with your smartphone c: The device cannot connect to your smartphone d: The neckloop has not been attached correctly to the device	a: Recharge the battery b: Pair the device with your smartphone c: Activate Bluetooth on your smartphone d: Attach the neckloop connectors correctly to the device
You cannot connect to your hearing care professional	Your smartphone is offline	Make sure that your smartphone is connected to the internet and try again

Problem	Possible cause	Solution
You have connected to your hearing care professional, but the connection to your hearing aid disappears	Other devices such as LED lamps, power supplies or adaptors might be interfering with the connection	Try switching off any device that could be interfering with the connection

If you still experience problems with connecting, pairing or using the device, contact your hearing care professional for help.

REGULATORY INFORMATION

EU Directives

Directive 2014/53/EU

Hereby, WSAUD A/S declares that the WIDEX REMOTE LINK device is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

The WIDEX REMOTE LINK contains radio transmitters operating at: 10.6 MHz, -54 dB μ A/m @10 m 2.4 GHz, 2.2 mW EIRP.

A copy of the Declaration of Conformity according to 2014/53/EU can be found at:

<https://global.widex.com/doc>

N26346

Information regarding disposal

Do not dispose of hearing aids, hearing aid accessories and batteries along with ordinary household waste.

Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for electrical waste and electronic equipment, or given to your hearing care professional for safe disposal.

SYMBOLS

Symbols commonly used by WSAUD A/S in medical device labelling (labels/IFU/etc.)

Symbol	Title/Description
--------	-------------------

Manufacturer

The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.

Catalogue number

The product's catalogue (item) number.
--

Consult instructions for use

The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.
--

Warning

Text marked with a warning symbol must be read before using the product.
--

WEEE mark

“Not for general waste”. When a product is to be discarded, it must be sent to a designated collection point for recycling and recovering to prevent the risk of harm to the environment or human health as a result of the presence of hazardous substances.

Symbol	Title/Description
	CE mark The product is in conformity with the requirements set out in European CE marking directives.
	RCM mark The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand markets.

FCC AND ISED STATEMENTS

FCC ID: TTY-PRLI

IC: 5676B-PRLI

CAN ICES-3(B)/NMB-3 (B)

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by WSAUD A/S could void the user's authority to operate the equipment.

ISED Statement / Déclaration d'ISED

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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ISED RADIATION EXPOSURE STATEMENT:

This equipment complies with ISED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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Version 2.0, January 2004

<http://apache.org/licences/>

TERMS AND CONDITIONS FOR USE, REPRODUCTION, AND DISTRIBUTION

1. Definitions.

"License" shall mean the terms and conditions for use, reproduction, and distribution as defined by Sections 1 through 9 of this document.

"Licensor" shall mean the copyright owner or entity authorized by the copyright owner that is granting the License.

"Legal Entity" shall mean the union of the acting entity and all other entities that control, are controlled by, or are under common control with that entity. For the purposes of this definition, "control" means (i) the power, direct or indirect, to cause the direction or management of such entity, whether by contract or otherwise, or (ii) ownership of fifty percent (50%) or more of the outstanding shares, or (iii) beneficial ownership of such entity.

"You" (or "Your") shall mean an individual or Legal Entity exercising permissions granted by this License.

"Source" form shall mean the preferred form for making modifications, including but not limited to software source code, documentation source, and configuration files.

"Object" form shall mean any form resulting from mechanical transformation or translation of a Source form, including but not limited to compiled object code, generated documentation, and conversions to other media types.

"Work" shall mean the work of authorship, whether in Source or Object form, made available under the License, as indicated by a copyright notice that is included in or attached to the work (an example is provided in the Appendix below).

"Derivative Works" shall mean any work, whether in Source or Object form, that is based on (or derived from) the Work and for which the editorial revisions, annotations, elaborations, or other modifications represent, as a whole, an original work of authorship. For the purposes of this License, Derivative Works shall not include works that remain separable from, or merely link (or bind by name) to the interfaces of, the Work and Derivative Works thereof.

"Contribution" shall mean any work of authorship, including the original version of the Work and any modifications or additions to that Work or Derivative Works thereof, that is intentionally submitted to Licensor for inclusion in the Work by the copyright owner or by an individual or Legal Entity authorized to submit on behalf of the copyright owner. For the purposes of this definition, "submitted" means any form of electronic, verbal, or written communication sent to the Licensor or its representatives, including but not limited to communication on electronic mailing lists, source code control systems, and issue tracking systems that are managed by, or on behalf of, the Licensor for the purpose of discussing and

improving the Work, but excluding communication that is conspicuously marked or otherwise designated in writing by the copyright owner as "Not a Contribution."

"Contributor" shall mean Licensor and any individual or Legal Entity on behalf of whom a Contribution has been received by Licensor and subsequently incorporated within the Work.

2. Grant of Copyright License.

Subject to the terms and conditions of this License, each Contributor hereby grants to You a perpetual, worldwide, non-exclusive, no-charge, royalty-free, irrevocable copyright license to reproduce, prepare Derivative Works of, publicly display, publicly perform, sublicense, and distribute the Work and such Derivative Works in Source or Object form.

3. Grant of Patent License.

Subject to the terms and conditions of this License, each Contributor hereby grants to You a perpetual, worldwide, non-exclusive, no-charge, royalty-free, irrevocable (except as stated in this section) patent license to make, have made, use, offer to sell, sell, import, and otherwise transfer the Work, where such license applies only to those patent claims licensable by such Contributor that are necessarily infringed by their Contribution(s) alone or by combination of their Contribution(s) with the Work to which such Contribution(s) was submitted. If You institute patent litigation against any entity (including a cross-claim or counterclaim in a lawsuit) alleging that the Work or a Contribution incorporated within the Work constitutes direct or contributory patent infringement, then any patent licenses granted to You under this License for that Work shall terminate as of the date such litigation is filed.

4. Redistribution.

You may reproduce and distribute copies of the Work or Derivative Works thereof in any medium, with or without modifications, and in Source or Object form, provided that You meet the following conditions:

1. You must give any other recipients of the Work or Derivative Works a copy of this License; and
2. You must cause any modified files to carry prominent notices stating that You changed the files; and
3. You must retain, in the Source form of any Derivative Works that You distribute, all copyright, patent, trademark, and attribution notices from the Source form of the Work, excluding those notices that do not pertain to any part of the Derivative Works; and
4. If the Work includes a "NOTICE" text file as part of its distribution, then any Derivative Works that You distribute must include a readable copy of the attribution notices contained within such NOTICE file, excluding those notices that do not pertain to any part of the Derivative Works, in at least one of the following places: within a NOTICE text file distributed as part of the Derivative Works; within the Source form or documentation, if provided along with the Derivative Works; or, within a display generated by the Derivative Works, if and wherever such third-party notices normally appear. The contents of the NOTICE file are for informational purposes only and do not modify the License. You may add Your own attribution notices within Derivative Works that You distribute, alongside or as an addendum to the NOTICE text from the Work, provided that such additional attribution notices cannot be construed as modifying the License.

You may add Your own copyright statement to Your modifications and may provide additional or different license terms and conditions for use, reproduction, or distribution of Your modifications, or for any such Derivative Works as a whole, provided Your use, reproduction, and distribution of the Work otherwise complies with the conditions stated in this License.

5. Submission of Contributions.

Unless You explicitly state otherwise, any Contribution intentionally submitted for inclusion in the Work by You to the Licensor shall be under the terms and conditions of this License, without any additional terms or conditions. Notwithstanding the above, nothing herein shall supersede or modify the terms of any separate license agreement you may have executed with Licensor regarding such Contributions.

6. Trademarks.

This License does not grant permission to use the trade names, trademarks, service marks, or product names of the Licensor, except as required for reasonable and customary use in describing the origin of the Work and reproducing the content of the NOTICE file.

7. Disclaimer of Warranty.

Unless required by applicable law or agreed to in writing, Licensor provides the Work (and each Contributor provides its Contributions) on an "AS IS" BASIS, WITHOUT WARRANTIES OR CONDITIONS OF ANY KIND, either express or implied, including, without limitation, any warranties or conditions of TITLE, NON-INFRINGEMENT, MERCHANTABILITY, or FITNESS FOR A PARTICULAR PURPOSE. You are solely re-

sponsible for determining the appropriateness of using or redistributing the Work and assume any risks associated with Your exercise of permissions under this License.

8. Limitation of Liability.

In no event and under no legal theory, whether in tort (including negligence), contract, or otherwise, unless required by applicable law (such as deliberate and grossly negligent acts) or agreed to in writing, shall any Contributor be liable to You for damages, including any direct, indirect, special, incidental, or consequential damages of any character arising as a result of this License or out of the use or inability to use the Work (including but not limited to damages for loss of goodwill, work stoppage, computer failure or malfunction, or any and all other commercial damages or losses), even if such Contributor has been advised of the possibility of such damages.

9. Accepting Warranty or Additional Liability.

While redistributing the Work or Derivative Works thereof, You may choose to offer, and charge a fee for, acceptance of support, warranty, indemnity, or other liability obligations and/or rights consistent with this License. However, in accepting such obligations, You may act only on Your own behalf and on Your sole responsibility, not on behalf of any other Contributor, and only if You agree to indemnify, defend, and hold each Contributor harmless for any liability incurred by, or claims asserted against, such Contributor by reason of your accepting any such warranty or additional liability.

END OF TERMS AND CONDITIONS

Appendix: How to apply the Apache License to your work

To apply the Apache License to your work, attach the following boiler-plate notice, with the fields enclosed by brackets "[]" replaced with your own identifying information. (Don't include the brackets!) The text should be enclosed in the appropriate comment syntax for the file format. We also recommend that a file or class name and description of purpose be included on the same "printed page" as the copyright notice for easier identification within third-party archives.

Copyright [yyyy] [name of copyright owner]

Licensed under the Apache License, Version 2.0 (the "License"); you may not use this file except in compliance with the License. You may obtain a copy of the License at

<http://www.apache.org/licenses/LICENSE-2.0>

Unless required by applicable law or agreed to in writing, software distributed under the License is distributed on an "AS IS" BASIS, WITHOUT WARRANTIES OR CONDITIONS OF ANY KIND, either express or implied. See the License for the specific language governing permissions and limitations under the License.

Copyright © Widex A/S, 2017 - 2018. All rights reserved. Any unauthorized redistribution or use of the software in source or binary form is prohibited and will be prosecuted when discovered.

Made for:

iPhone XS, iPhone XR, iPhone 8

iPad (6th generation), iPad mini 4, iPad Pro

Use of the Made for badge means that an accessory has been designed to connect specifically to the Apple product(s) shown in the badge and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards.

iPad, iPad Air, iPad Pro and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. The trademark "iPhone" is used in Japan with a license from Aiphone K.K.